

Clinician Evaluation Guide

Svelte Premium Continence Wear — Clinician Evaluation Sample Pack

2 wears per size: M×2, L×2

Why Svelte

Svelte is designed in Australia and crafted in Japan for mobile and semi-mobile adults seeking discreet, underwear-like continence support.

It is designed to support comfort, mobility and day-to-day wearability through breathable construction, rapid intake, even fluid distribution, core stability and adaptive fit during movement and sitting.

Product at a glance

- Fully breathable microporous backsheet using a single, continuous material extending through to the waist
- Soft topsheet, dermatologically tested for skin contact, fragrance-free and lotion-free
- Not made with natural rubber latex
- Rapid-intake absorbent core designed to draw in and disperse liquid quickly
- Helps reduce rewet and support a drier-feel wear experience between changes
- Odour management included as a secondary product support feature during wear
- Engineered core-support structure designed to support garment stability during movement and sitting
- Adaptive stretch waistband designed to flex with the body to support comfort and fit during movement, sitting and bending
- Size-specific containment support: soft, raised, flexible 3D leak guards with an adaptive fit during movement, sitting and position changes

M: wider leg-gather construction

L: contoured leg design for fuller thigh and hip profiles

- Slim, discreet, low-rustle construction for daily wear and user acceptance
- May support light overnight use when changed as required, depending on output, fit, skin condition and change routine. Svelte is not intended for extended single-garment overnight wear or higher-volume overnight needs.

Construction note

Svelte uses a single, continuous fully breathable backsheet material that extends from the absorbent zone through to the waist.

During evaluation, clinicians may note thermal comfort, waistband behaviour, outer surface feel and garment stability during sitting, movement and routine wear.

Best suited for:

- Mobile and semi-mobile adults seeking a pull-up for daytime use, with possible light overnight use when changed as required.
- Clients who have found heat, humidity, bulk, stiffness, noise or fit uncomfortable in some other products.
- Clients who prefer discreet, underwear-like support for community access, work or travel.
- Individuals being assessed for fit, comfort, rewet feel, core stability, garment stability and user acceptance during routine wear.

Not intended as the sole product for:

- Regular or unpredictable faecal incontinence, where tab-style briefs or bowel containment products may be more appropriate.
- Bedbound or fully immobile clients requiring 24-hour high-dependency care.
- Extended, single-garment overnight wear.
- Higher-volume overnight needs where changes are not possible.

Fit check: 60 seconds

- Confirm correct front and back orientation before assessing fit

- Confirm size using the waist and hip range
- Check adaptive fit during movement: walking, sitting, sit-to-stand and bending
- Confirm leg seal and leak guard position after movement
- Check waistband comfort, rolling, pressure and garment position after sitting

Size	Waist & Hip Range	Real-world working capacity
M	75–130 cm	~500 mL
L	100–140 cm	~500 mL

Sizing note (evaluation samples):

Evaluation samples are supplied in Australian sizing. Please assess fit using the stated waist and hip range for each size.

Capacity note:

Real-world working capacity is intended to reflect practical in-use absorbency more closely than maximum static laboratory absorbency figures alone.

Svelte uses a real-world working capacity of approximately 500 mL. This figure should be understood as a conservative working guide rather than a maximum absorbency claim or fixed wear-time guarantee.

Real-world performance may vary depending on void volume, frequency, fit, movement, sitting time, compression and change routine.

Trial protocol

- Trial each size twice where clinically relevant.
- Use during normal routine, including home or community settings where relevant.
- Across wears, include one extended sitting period and one active period.
- Observe at change: skin response, garment stability, core positioning, rewet feel and odour perception.
- Evaluation samples are supplied to support clinician-led assessment and feedback; quantities are limited (while stocks last).
- Evaluation kits are supplied for clinician or service-led assessment and are not for general consumer distribution.
- For further clinical evaluation, contact clinicians@getsvelte.com.au.

Key observations during evaluation

When evaluating Svelte alongside other pull-up or pad-based systems, clinicians may wish to observe:

- whether the absorbent core remains **evenly distributed** after wetting and movement
- whether the **outer backsheet or clothing shows any dampness** between changes
- whether **rewet feel is perceived** during sitting, activity or compression
- whether urine appears **distributed through the core** or **concentrated centrally**
- whether **localised swelling** or “pillowing” is observed
- whether the core remains **stable and well-positioned** during movement
- whether **twisting, bunching** or **collapse** is observed
- user comfort, tolerance and preference during routine wear

These observations may assist clinician-led comparison of rewet feel, comfort, garment stability and user acceptance between products.

Svelte

Rate versus the current product

(tick one option for each outcome area)

Outcome area	Better	Same	Worse	N/A
Heat build-up / sweating	☐	☐	☐	☐
Rewet feel	☐	☐	☐	☐
Skin comfort / irritation signals	☐	☐	☐	☐
Leakage experience during movement / position change	☐	☐	☐	☐
Odour control	☐	☐	☐	☐
Bulk / discretion under clothing	☐	☐	☐	☐
Noise / rustle during movement	☐	☐	☐	☐
Waist comfort / stability (sitting/bending)	☐	☐	☐	☐
Core stability / structural integrity (twisting, bunching, collapse)	☐	☐	☐	☐
User acceptance / preference	☐	☐	☐	☐

Contact & clinical support

For clinical resources, product information, fit guidance or professional product enquiries, contact:

clinicians@getsvelte.com.au

For NDIS ordering, plan manager invoices or payment documentation, please use the Plan Manager / NDIS administrative pathway or contact:

info@getsvelte.com.au

We welcome clinician feedback to support product evaluation, continence assessment discussions and appropriate product selection within clinician-led care.